



Make sure you have the ProConnect code provided by your healthcare provider to share your Glooko data



If you have any questions or concerns regarding Omnipod 5, please contact Omnipod Product Support Team at:

**1-855-POD-INFO Option 1
(1-855-763-4636)**

If you are experiencing any issues with Glooko, please contact Glooko Support:

1-800-206-6601

**www.glooko.com
support@glooko.com**



©2025 Insulet Corporation. Omnipod and the Omnipod logo are trademarks or registered trademarks of Insulet Corporation in the United States of America and other various jurisdictions. All rights reserved. The Bluetooth wordmark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Insulet Corporation is under license. Glooko is a trademark of Glooko, Inc. and used with permission. All other trademarks are the property of their respective owners. The use of third-party trademarks does not constitute an endorsement or imply a relationship or other affiliation. Insulet Canada Corporation 1540 Cornwall Rd #201, Oakville, ON L6J 7W5. INS-OHS-10-2024-00256V1.0




Omnipod® 5 & Glooko® Key Information

**An Omnipod 5 Leaflet for People
Living with Type 1 Diabetes**



LINK your Omnipod 5 Automated Insulin Delivery System data with Glooko

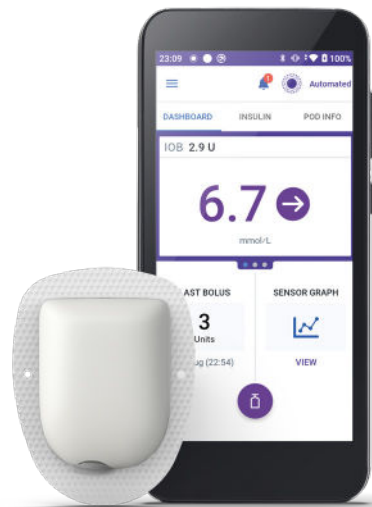

If you have missed linking your Omnipod 5 and Glooko, please follow these four simple steps:

1. Log into your online Omnipod® account at www.omnipod.com, using your Omnipod ID
2. Go to the **My Account** tab and then select **Account Linking** from the drop down menu
3. Select **sign up** or **log in** with Glooko
4. Proceed to Glooko and complete the process to link your account

When complete, your status will update to **Linked** and data will automatically upload into your personal Glooko account* which can be unlinked at any time to stop sharing data between Omnipod and Glooko.

*Must be connected to WiFi or cellular data, and Omnipod 5 users must have WiFi or cellular data access to receive updates in Glooko.

NOTE: All screen images are examples, for instructional purposes only.

Choose to **SHARE** your Omnipod 5 Automated Insulin Delivery System data with your healthcare provider



Entering a ProConnect code in Glooko is the only way for your healthcare provider to see your Omnipod 5 data on their computer.

ProConnect Code:
Provided by your
healthcare provider


If you have missed linking your Glooko account to your healthcare provider, please follow these five simple steps:

1. Log into your Glooko account at my.glooko.com
2. Select **Settings** from the drop-down menu (top right on the home screen)
3. Scroll down to Account and select **+ Add New Code**
4. Select **Continue**
5. Enter ProConnect Code then select **Submit**

NOTE: All screen images are examples, for instructional purposes only.

Using the GLOOKO MOBILE APP

You have the choice to use the
Glooko mobile app or the online
Glooko platform my.glooko.com

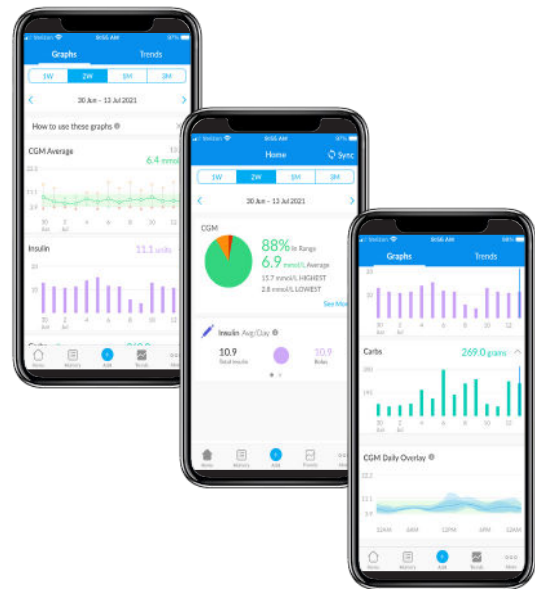

NOTE: All screen images are examples, for instructional purposes only.

For iOS



Download the app and log in

- From the App Store search for and install the **Glooko mobile app**
- Log in or create an account by filling out the requested information

View your data

Once your data is uploaded to Glooko, you can view your trends and graphs on your iPhone or iPad

Choose to share data with your healthcare provider

- Click the **More** tab
- Click **Share Reports** to email or print the reports

For Android



Download the app and log in


- From the Google Play™ Store search for and install the **Glooko mobile app**
- Log in or create an account by filling out the requested information

Sync and view your data

- Tap **Sync** from the Home Screen, and select **Omnipod 5 Automated Insulin Delivery System**
- Follow the prompts to complete the sync

Choose to share data with your healthcare provider

Click **Share Reports** to email or print the reports

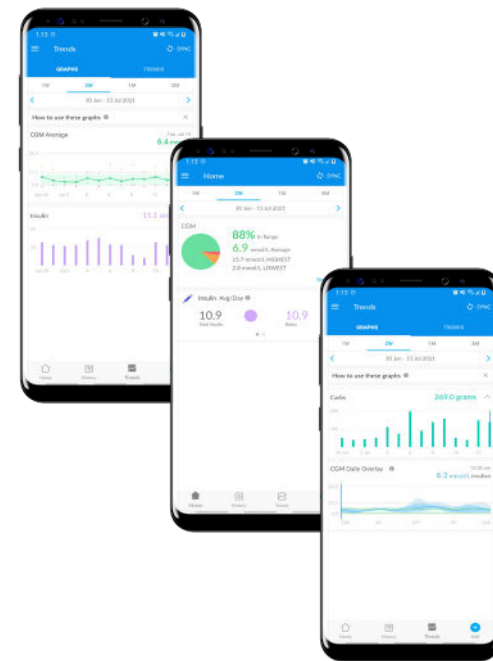

NOTE: All screen images are examples, for instructional purposes only.

Create, Store, or Print a PDF REPORT

You can create and view reports on the **Glooko mobile app** or at **my.glooko.com**

1. **Log in** to your account

2. **To create a report:**


- Click on your **Profile**
- Click **Create PDF Report** (top right)
- Choose desired reports and time frame and click **Create PDF**

3. **To save/print a report:**

Once the report is created, either save it to your desktop or a desired location, or print directly

TIP! Save Customized Report Sets:

- Click **Settings**
- Scroll down and click **New Favourite**
- Select desired reports, and name the report set (i.e. *Dr. Smith reports*)

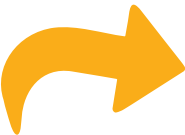

Great reports to get started:


Summary reports, Daily, and Weekly

Snapshot of **key statistics** and **aggregated information** to identify positives and challenges to support your care



The Device Settings Report

When your data is uploaded to Insulet Provided Glooko, your pump settings will be saved and this report can be a great resource for you and your care team

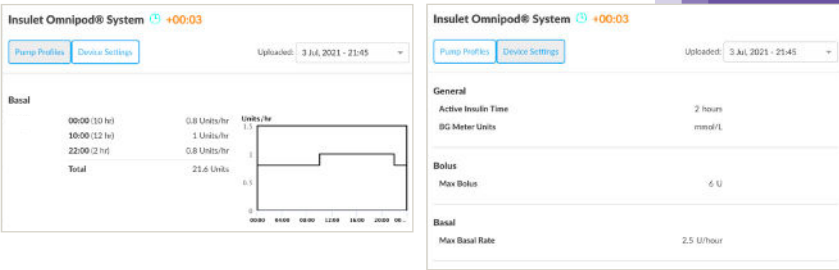

NOTE: All screen images are examples, for instructional purposes only.

NOTE: All screen images are examples, for instructional purposes only.